•• CONTRAINDICATIONS

A history of the patient indicating allergic reactions, especially in the methacrylate resin, or any other component of the dental material.

•• COLORS

- Green-Fluo; Pink-Fluo; Purple-Fluo; Lime-Fluo; Orange-Fluo; Blue-Fluo; Yellow-Fluo
- Green-Pearl; Pink-Pearl; Blue-Pearl; Golden-Pearl
- Clear-Red; White-Pearl

•• PRECAUTIONERY INFORMATION

Jen-Rainbow contains methacrylic resins that may cause an allergic reaction. Avoid long or repeating contact of not-polymerized material with the skin (allergic contact dermatitis is possible), soft tissue of the oral cavity and eyes.

In case of contact, immediately wash thoroughly a place of contact with water and soap. If there is rash or other signs of allergic reactions on the skin stop using the material and ask for medical care.

Using of this product on patients with allergies on acrylic materials is to be avoided.

•• PRECAUTIONARY MEASURES

- While working with the material use special glasses, gloves, clothes and mask. Goggles are recommended for the patients also.
- Observe security measures provided for the work with powdery, siliceous substances. Use goggles, a mask and gloves.

•• COLLATERAL REACTIONS

The product can cause irritation of eyes, skin and mucous membranes. (see. CAUTION section).

•• INTERACTION WITH OTHER DENTAL MATERIALS

Using of eugenol-containing materials in a combination with Jen-Rainbow is contraindicated. Eugenol-containing dental materials can have negative impact on ability of polymerization.

•• WARRANTY

The manufacturer warrants the quality of manufactured products. The adverse events inflicted by violation of user manual, storage conditions and other events inflicted by non-stipulated usage of the material are not the subjects of warranty.

The customer is responsible for determination of suitability of this product for user’s application. Warranty conditions: the product does not comply with requirements declared by manufacturer. In this case the manufacturer replaces the defective material within warranty period.

•• LIMITATION OF LIABILITY

The manufacturer’s liability is limited by only cases stipulated by direct legislation of the country.

•• STORAGE

- Store at room temperature.
- Shelf life is 3 years.
- Refrigeration will further increase shelf life.
- DO NOT FREEZE!

•• RECYCLING

Dispose of the medical device in accordance with local / regional / national / international legal requirements.

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