


# NOW AVAILABLE 1.2 ML PREFILLED SYRINGES

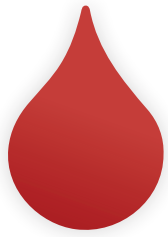

**ASTRINGEDENT™ X**  
— HEMOSTATIC AGENT —

Achieving hemostasis quickly is vital for efficient tissue management. That's why so many dentists choose Astringedent X hemostatic for every case. Now available in prefilled 1.2 ml syringes, Astringedent X hemostatic is faster and easier to use than ever before. With the Metal Dento-Infusor™ tip and the proper scrubbing technique, Astringedent X hemostatic stops minor to severe capillary bleeding in seconds, delivering profound hemostasis and consistent results time after time.<sup>1</sup>

# Astringedent™ X

ASTRINGEDENT X HEMOSTATIC IS YOUR FIRST STEP IN ACHIEVING GREAT FITTING CROWNS, BRIDGES, VENEERS, AND TEMPORARIES. IT IS AVAILABLE IN 30 ML BOTTLES, INDISPENSE™ SYRINGES, AND NOW PREFILLED 1.2 ML SYRINGES.



Bleeding at preparation.



Astringedent X hemostatic is rubbed firmly against bleeding tissue.



When bleeding stops, give a firm air/water spray to test for hemostasis and remove coagulum.




Profound hemostasis.

- THE PREMIER COAGULATIVE HEMOSTATIC FOR QUICKLY ACHIEVING HEMOSTASIS AND CONTROL OF SULCULAR FLUIDS
- 12.7% IRON SOLUTION

- ULTRADENT'S FASTEST AND MOST POWERFUL HEMOSTATIC<sup>2</sup>
- RECOMMENDED FOR ALL CASES, INCLUDING THE MOST DIFFICULT-TO-STOP, PROBLEM BLEEDING



**5881 - Astringedent X Syringe 20pk**  
20 x 1.2 ml syringes

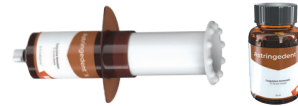

**112 - Astringedent X Bottle 1pk**  
**690 - Astringedent X IndiSpense Syringe 1pk**  
30 ml



The Metal Dento-Infusor™ tip with Comfort Hub™ design has a metal, bent cannula with a flared brush padded end. The larger ergonomic wings provide a secure, comfortable grasp of the tip.

LOK-TITE™ + COMFORT HUB™	100pk	500pk
19 ga Metal Dento-Infusor	4955	4956



"It was very fast-acting and controlled bleeding quite well" and "Hemostasis was immediate! The product was excellent."



1. Bandi M, Mallineni SK, Nuvvula S. Clinical applications of ferric sulfate in dentistry: A narrative review. *J Cons Dent*. 2017 Jul-Aug;20(4):278-281. DOI: 10.4103/jcd.jcd\_259\_16. 2. Data on file.